# KX06

## User Guide

KX06V2

EN	User Guide	2
FR	Guide de l'utilisateur	12
DE	Bedienungsanleitung	22
IT	Guida per il paziente	32
ES	Guía del usuario	42
NL	Gebruikershandleiding	52
PL	Podręcznik użytkownika	62
PT	Guia do utilizador	72
CS	Návod k obsluze	82

**KX06V2**

**Please keep these instructions.**

# Contents

Contents .....	2
1 Description and Intended Purpose .....	3
2 Safety Information .....	4
3 Construction .....	5
4 Maintenance .....	6
5 Limitations on Use .....	7
6 Removing the Cosmetic Cover .....	7
7 Adjusting Swing Resistance .....	8
7.1 Flexion Resistance .....	8
7.2 Extension Resistance .....	9
7.3 Stirrup Function and Features .....	9
8 Fitting the Cosmetic Cover .....	10

---

# 1 Description and Intended Purpose

These instructions are for the user.

The term *device* is used throughout these instructions for use to refer to KX06V2.

## Application

This device is a knee unit that is for use exclusively as a component of a lower limb prosthesis.

This device is a polycentric knee unit that uses both a 4-bar mechanism and a hydraulic cylinder to control the phases of the gait cycle. It provides swing and stance control for active hip disarticulation, transfemoral and knee disarticulation users.

The geometry of the device provides increased toe clearance during swing phase. The build height is short and upon sitting minimizes protrusion of the prosthetic knee beyond the contralateral limb. This combination of features reduces asymmetry for users with longer residual limbs.

This device is manufactured from water resistant materials and is designed to accommodate limited exposure to such environments.

## Features

- Adjustable stance support
- Adjustable swing (flexion and extension) resistances
- Reduced build height\*
- Increased toe clearance during swing phase\*
- Reduced protrusion during sitting\*
- Flexion Lock mode
- Cycling mode

\*Compared to a single axis knee unit.

Ensure you have had gait training before using this device.

Make sure that you understand all instructions for use, drawing particular attention to all maintenance and safety information sections.

This device is for a single user.

## 2 Safety Information

**This warning symbol highlights important safety information which must be followed carefully.**

**1. Be aware of finger trap hazard at all times.**

**2. Any changes in the performance or function of the limb e.g. instability, double-action, restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.**

**3. Always use a hand rail when descending stairs and at any other time if available.**

**4. Any excessive changes in heel height after finalization of alignment may adversely affect limb function.**

**5. After continuous use the cylinder can become hot.**

**6. The device is not designed for prolonged submersion. Ensure any use of the device in water complies with the conditions given in "Limitations on Use" (see Section 5).**

**7. After any period of immersion in water, rinse the device with fresh clean water and then thoroughly dry before reuse.**

**8. Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.**

**9. Care should be taken when carrying heavy loads as this may adversely affect the stability of the device.**

**10. Contact your practitioner if your condition changes.**

**11. Avoid exposure to extreme heat and/or cold**

**12. Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.**

**13. Do not adjust the swing resistance settings set by your clinician, without prior agreement and training from your clinician.**

**14. If you adjust swing resistance to settings unsuitable for your condition and required operation, you risk injury from tripping, loss of balance and falling.**

**15. Whenever the device's swing resistance is adjusted, caution should be exercised until you become fully accustomed to the new settings to minimise the risk of injury from tripping, loss of balance and falling.**


- Notes:
- i) Always make a note of the swing resistance settings set by your clinician before making any swing resistance adjustments, so you can easily return the device to these settings as required.
  - ii) If you remove the cosmetic cover to make swing resistance adjustments, take care to avoid finger trap hazards while the cover is removed.
  - iii) When adjustments are completed, immediately refit the cosmetic cover (see Section 8) to help protect against finger trap hazards in use.

# 3 Construction

## Principal Parts

- Frame Aluminium alloy, brass, stainless steel, titanium, steel
- Knee control Various materials, principally aluminium alloy, stainless steel, acetal homopolymer, nitrile butadiene rubber (NBR), hydraulic fluid
- Knee pad Polyurethane
- Cosmetic cover Thermoplastic elastomer

## Component Identification

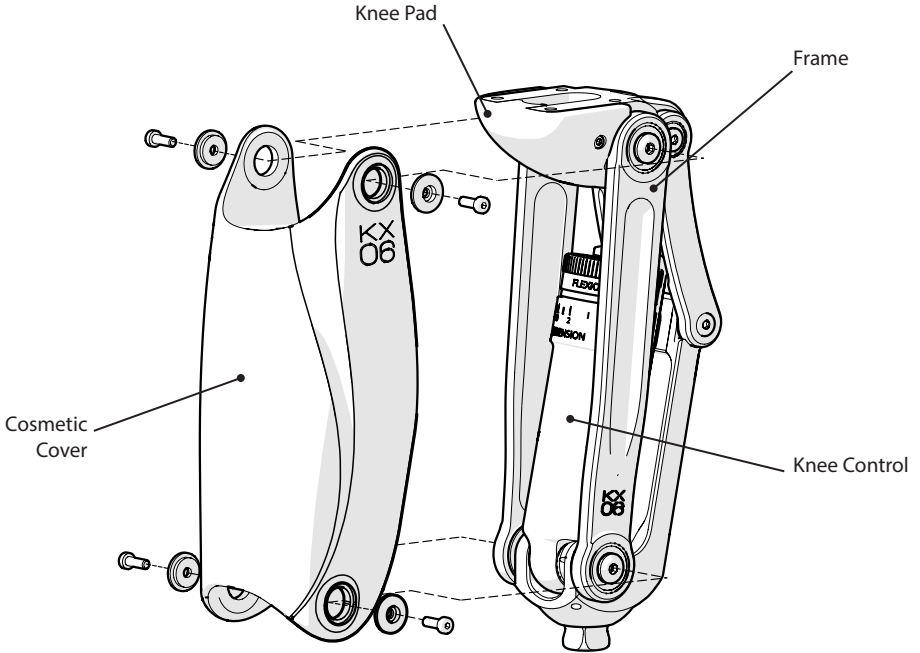---

## 4 Maintenance

An annual visual inspection is recommended. Check for visual defects that may affect proper function.

Maintenance must be carried out by competent personnel.

Do not carry out maintenance on this device yourself. Instead, return it to a Blatchford center for maintenance. If the device is still under warranty, we will loan you another knee unit while we carry out maintenance.

Report any changes in performance of this device to the practitioner. Changes in performance may include:

- Increase or decrease in knee stiffness
- Instability
- Reduced knee support (free movement)
- Any unusual noise\*

\* Due to the nature of the hydraulics there may be some slight air noise from the cylinder during the first few steps. This is not detrimental to the function of the unit and should dissipate quickly.

If symptoms persist, please consult your practitioner.

### **Storage and Handling**

When storing for prolonged periods place the device vertically with trunnion uppermost. Alternatively and in addition, air management may be assisted by cycling the knee control several times with the stance resistance switched off (refer to Section 7.3).

Use product packaging supplied.

### **Cleaning**

Use a damp cloth and mild soap to clean the outside surfaces.

DO NOT use aggressive cleansers.

## 5 Limitations on Use

### Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

The device is suitable for submersion in fresh water only.

Knee must be locked while submerged, then rinsed and thoroughly dried.

Light surface corrosion affects neither the function nor the security of this device.

However, if it is heavily corroded, stop using and contact your practitioner.

Avoid exposing the device to abrasive or corrosive environments, such as those containing sand, salt or chemically treated water e.g. swimming pools. After any exposure to such environments rinse the device in fresh water and dry thoroughly.

Exclusively for use between -10 °C and 50 °C  
(-14 °F and 122 °F).

It is recommended that only Blatchford products be used in conjunction with the device.

Suitable for submersion


## 6 Removing the Cosmetic Cover

If you need to adjust swing resistance, first remove the cosmetic cover to access the adjustment cap.

To remove the cosmetic cover:

1. Using a 2.5 mm hex key, unscrew, remove and set aside 4 x M4 hex screws and 4 x spacers that fasten the cosmetic cover to the chassis anterior pivot and carrier anterior pivot.
2. Pull off the cosmetic cover and set aside.


 Be aware of finger trap hazards with the cosmetic cover removed.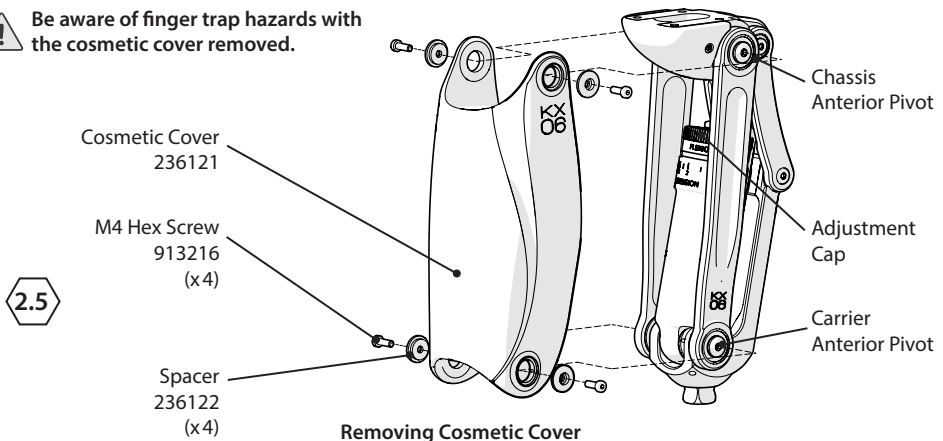

# 7 Adjusting Swing Resistance

## 7.1 Flexion Resistance

To access the adjustment cap, first remove the cosmetic cover (see Section 6).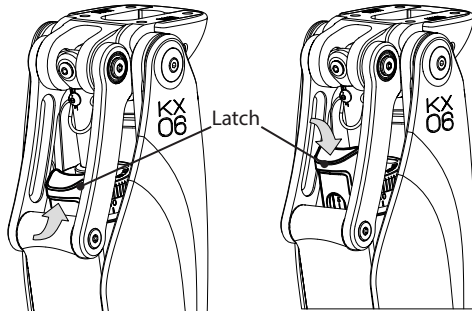

Raise latch to allow adjustment cap to turn.


Lower latch to lock adjustment cap.

Note... If you cannot turn the adjustment cap with your fingers, use a hydraulic-knee-control adjuster (940091).

**!** Take care not to force the adjuster ring and overcome the adjustment stops; this could cause loss of function.

1


- 1) If there is excessive heel rise: Increase flexion resistance.
- 2) If there is insufficient heel rise: Decrease flexion resistance.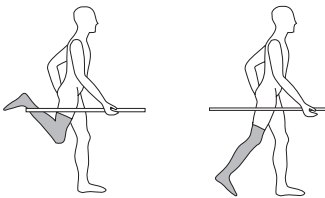


**!** Warning! If adjusting the flexion resistance has no effect on heel rise, check stirrup is in down position and ensure that the user is initiating flexion whilst still loading the toe.

Note that a hyperextension moment is needed about the knee to initiate flexion.

**!** Read "Safety Information", items 13 to 15 and Notes i to iii on page 4 before proceeding.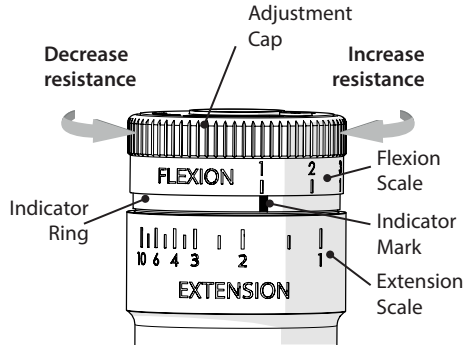


Hydraulic-Knee-Control Adjuster (940091)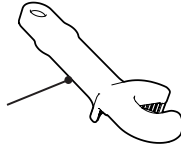

Note... The same adjustment cap is used to adjust both flexion resistance and extension resistance.

The flexion scale on the adjustment cap is numbered from left to right 1 (minimum) to 10 (maximum). Directly below the flexion adjustment cap is an indicator mark.

Turning the cap counterclockwise reduces flexion resistance until the minimum resistance is reached where "1" on the flexion scale is over the indicator mark.

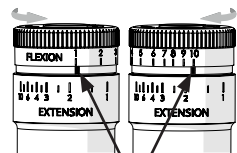Indicator Mark  
Minimum Flexion (1)      Maximum Flexion (10)

Turning the cap clockwise increases the resistance to a maximum setting at "10".

To adjust flexion resistance:

- 1) Raise the latch.
- 2) Turn the adjustment cap to the required flexion setting.
- 3) Lower the latch to lock the cap and prevent accidental adjustment.


## 7.2 Extension Resistance

To access the adjustment cap, first remove the cosmetic cover (see Section 6).

**2**

- 1) If there is excessive terminal impact on knee extension: **increase** resistance.
- 2) If the knee does not extend satisfactorily: **decrease** resistance.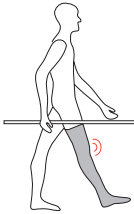

Note... As a 'rule of thumb' flexion resistance should be greater than extension resistance.


Note... If there is any doubt over the position of the indicator ring in relation to the settings, its position can be reset by turning the adjustment cap to maximum flexion, then maximum extension prior to making any adjustments.

After completing all flexion and extension resistance adjustments, refit the cosmetic cover if previously removed (see Section 8).

**Make a note of the flexion setting before starting to adjust the extension resistance.**

To increase extension resistance:

- 1) Raise the latch and turn the adjustment cap in the clockwise direction until it reaches 10 on the flexion scale.
- 2) Continue turning the cap clockwise to engage and turn the indicator ring clockwise; this moves the indicator mark from right to left (clockwise), increasing extension resistance from 1 to 10 on the extension scale.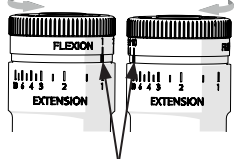

Indicator Mark

Minimum Extension (1)      Maximum Extension (10)

To reduce extension resistance:

- 3) Raise the latch and turn the adjustment cap anticlockwise to minimum on the flexion scale then continue turning until the desired extension resistance is reached on the extension scale.


When the correct extension resistance setting is achieved, turn the adjustment cap back to the previously noted flexion resistance setting, taking care not to re-engage and turn the indicator ring again.

Lower the latch to lock the cap and prevent accidental adjustment.

## 7.3 Stirrup Function and Features


**How to switch off stance resistance (for cycling etc.)**

- 1) Load toe of prosthesis.
- 2) While toe is loaded, lift stirrup up.
- 3) Stance now off but knee may be unstable.
- 4) Lower stirrup to resume normal operation.

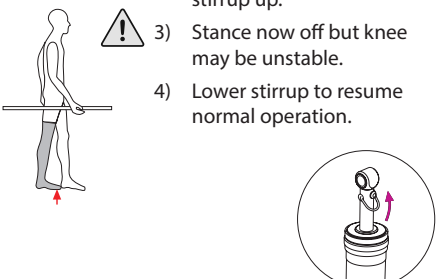**How to lock knee against flexion (for prolonged standing etc.)**


- 1) Flex knee slightly without overriding stance resistance.
- 2) Lift stirrup.
- 3) The knee is now locked against flexion but will extend.
- 4) Lower stirrup to resume normal operation.

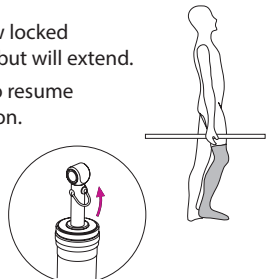## 8 Fitting the Cosmetic Cover

 **Always fit the cosmetic cover after all adjustments have been made to help prevent finger trap hazards.**

1. Fit the cosmetic cover to the device at the chassis anterior pivot and carrier anterior pivot using the 4 x M4 hex screws and 4 x spacers supplied (see figure below).
2. Tighten the screws using the 2.5 mm hex key supplied. Avoid over-tightening the screws.

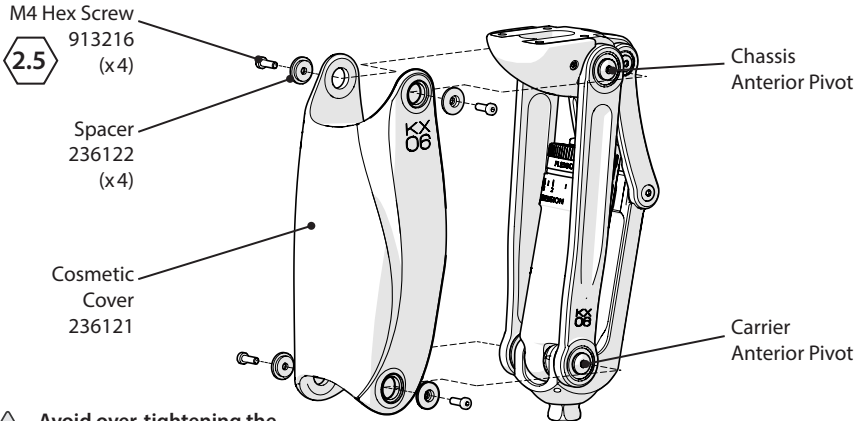 **Avoid over-tightening the cover screws**

Fitting Cosmetic Cover

## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient – multiple use

## Warranty

This device is warranted for 36 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See the Blatchford website for the current full warranty statement.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

This product contains hydraulic oil, mixed metals and plastics. It should be recycled where possible in accordance with local waste recycling regulations.

## Trademark Acknowledgements

KX06 and Blatchford are registered trademarks of Blatchford Products Limited.

## Manufacturer's Registered Address

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**Blatchford Europe GmbH**
**Endolite India Ltd.**