# Javelin

## Instructions for Use

JAV22L1S–JAV30R9S  
JAV22L1SD–JAV30R9SD

|    |                        |     |
|----|------------------------|-----|
| EN | Instructions for Use   | 2   |
| SL | Navodila za uporabo    | 19  |
| BG | Инструкции за употреба | 36  |
| HR | Upute za uporabu       | 53  |
| SK | Návod na použitie      | 70  |
| HU | Használati útmutató    | 87  |
| EL | Οδηγίες χρήσης         | 104 |
| LV | Lietošanas pamācība    | 121 |
| LT | Naudojimo instrukcija  | 138 |
| ET | Kasutusjuhend          | 155 |

# Contents

|   |    |
|---|----|
| Contents .....  | 2  |
| 1 Description and Intended Purpose .....              | 3  |
| 2 Safety Information .....                            | 5  |
| 3 Construction .....                                  | 6  |
| 4 Function.....                                       | 7  |
| 5 Maintenance .....                                   | 7  |
| 6 Limitations on Use .....                            | 8  |
| 7 Bench Alignment.....                                | 9  |
| 7.1 Static Alignment.....                             | 9  |
| 7.2 Dynamic Alignment .....                           | 9  |
| 7.3 Transfemoral Alignment.....                       | 9  |
| 8 Cutting Pylon to Length .....                       | 10 |
| 9 Foot Shell Removal & Pylon/Spring Replacement ..... | 12 |
| 10 Fitting Advice.....                                | 15 |
| 11 Technical Data .....                               | 16 |
| 12 Ordering Information .....                         | 17 |

---

# 1 Description and Intended Purpose

These Instructions for Use are intended for the practitioner and user unless otherwise stated.

The term *device* is used throughout to refer to the Javelin.

Please read and ensure you understand these instructions, in particular all safety information and maintenance instructions.

## Application

This device is to be used exclusively as part of a lower limb prostheses, intended for a single user. A lightweight, high-energy-return foot, ideal for medium impact use. The flexible extended pylon reduces shock load transmission making it suitable for work and leisure activities. Independent heel and toe springs provide some axial deflection. The split toe provides good ground compliance.

## Features

- Small sizes available
- E-carbon composite foot with independent springs
- Heel and toe spring sets available in a range on nine spring ratings from low (1) to high (9) impact
- Dynamic pylon enhances shock absorption
- Choice of male or female sliding/rotating housings available separately
- Sandal toe foot shell and glide sock included

## Activity Level

This device is recommended for users that have the potential to achieve Activity Level 3 (weight limits apply, see *Technical Data*). Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances and any such decision should be made with sound and thorough justification.

### Activity Level 1

Has the ability or potential to use a prosthesis for transfers or ambulation on level surfaces at fixed cadence. Typical of the limited and unlimited household ambulator.

### Activity Level 2

Has the ability or potential for ambulation with the ability to traverse low-level environmental barriers such as curbs, stairs, or uneven surfaces. Typical of the limited community ambulator.

### Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

### Activity Level 4

Has the ability or potential for prosthetic ambulation that exceeds basic ambulation skills, exhibiting high impact, stress, or energy levels. Typical of the prosthetic demands of the child, active adult, or athlete.

There may also be a number of users in the Activity Levels 2 and 4\* who would benefit from the smoother transition from heel to toe offered by this device, but this decision should be made with sound and thorough justification.

\*Maximum user weight 100 kg and always use one higher spring rate category than shown in the Spring set selection table.

### Contraindications

This device may not be suitable for Activity Level 1 individuals and is not recommended for competitive sports events, as these types of users will be better served by a specially designed prosthesis optimized for their needs.

### Clinical Benefits

- High level of stability during midstance, compared to other energy-storage-and-return feet and other non-energy-storage-and-return feet
- Improved energy return from springs, compared to other energy-storage-and-return feet
- Improved mediolateral ground compliance, compared to feet without a split toe spring

### Spring Set Selection

|          | User Weight        |                    |                    |                    |                    |                     |                      |                      |                      |                      |                 |
|----------|--------------------|--------------------|--------------------|--------------------|--------------------|---------------------|----------------------|----------------------|----------------------|----------------------|-----------------|
| Activity | 44-52<br>(100-115) | 53-59<br>(116-130) | 60-68<br>(131-150) | 69-77<br>(151-170) | 78-88<br>(171-195) | 89-100<br>(196-220) | 101-116<br>(221-255) | 117-130<br>(256-285) | 131-147<br>(286-325) | 148-166<br>(326-365) | kg (lb)         |
| 3        | 1                  | 1                  | 2                  | 3                  | 4                  | 5                   | 6                    | 7                    | 8                    | 9                    | Foot spring set |

Notes:

If in doubt choosing between two categories, choose the higher rate spring set.

Foot Spring set recommendations shown are for transtibial users.

For transfemoral users we suggest selecting a spring set one category lower, refer to *Fitting Advice* section to ensure satisfactory function and range of movement.

## 2 Safety Information

This warning symbol highlights important safety information which must be followed carefully.

Any changes in the performance or function of the limb e.g. restricted or excessive movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

The device is not suitable for extreme sports, competitive running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling or running is acceptable.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified practitioner.

The user should be advised to contact their practitioner if their condition changes.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

To minimize the risk of slipping and tripping, appropriate footwear that fits securely onto the foot shell must be used at all times.

The device is designed for prolonged submersion and suitable for immersion in fresh water only. Ensure any use of the device in water complies with the conditions given in *Limitations on Use*.

To minimize the risk of slipping and tripping, appropriate footwear that fits securely onto the foot shell must be used at all times.

Be aware of finger trap hazard at all times.

**WARNING:** Carbon fiber dust is non-toxic but is an irritant to the skin, eyes and lungs. Always read and comply with local and national Health & Safety regulations regarding the cutting of carbon fiber.

When cutting carbon fibre:


1. Ensure working area is well ventilated and/or under extraction.
2. Wear eye protection/face shield, dust mask and protective gloves.

# 3 Construction

## Principal Parts

- Carrier Aluminum
- Heel & Toe Springs E-carbon composite
- Spring Attachment Screws Titanium
- Glide Sock Ultra-high molecular weight polyethylene (UHM PE)
- Foot Shell Polyurethane (PU)

## Component Identification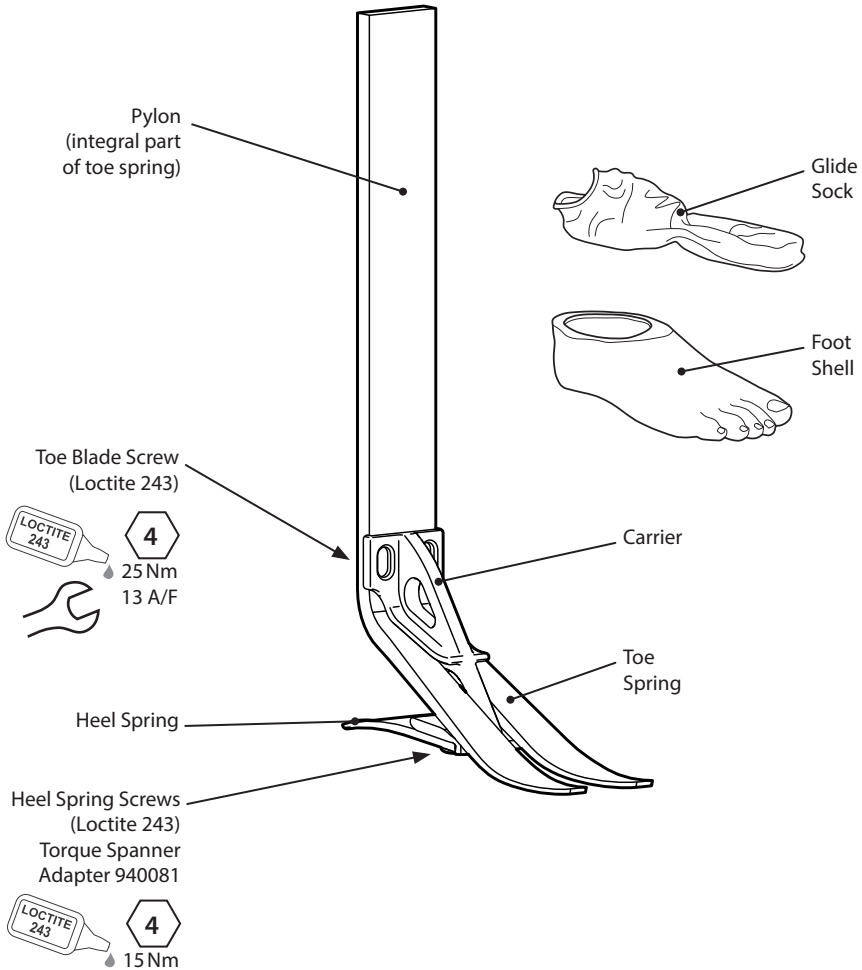

---

## 4 Function

This device comprises an e-carbon composite toe spring pylon and independent heel spring. The pylon should be cut to length to suit individual users.

Heel and toe springs are attached to the carrier assembly using titanium screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

The device requires either a male or female top housing, supplied separately, to be fitted to the shin to complete the prosthetic limb and provide connection to the socket.

---

## 5 Maintenance

Visually check the device regularly.

Report any changes in performance of this device to the practitioner/service provider e.g. unusual noises, increased stiffness, restricted/excessive movement, significant wear, corrosion or excessive discoloration from long term exposure to UV.

Inform the practitioner/service provider of any changes in body weight and/or activity level.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces. DO NOT use aggressive cleansers.

***The remaining instructions in this section are for practitioner use only.***

This maintenance must be carried out only by competent personnel (practitioner or suitable trained technician).

The following routine maintenance is to be carried out at least annually:

- Remove foot shell and glide sock, check for damage or wear and replace if necessary.
- Check all screws for tightness, see *Construction* section; clean and reassemble as necessary.
- Check heel and toe springs for signs of delamination or wear and replace if necessary. Some minor surface damage may occur after a period of use, this does not affect the function or strength of the foot.

Ensure the user has read and understood all safety and user-level maintenance information.

Advise the user that a regular visual check of the device is recommended and signs of wear that may affect function should be reported to their service provider (e.g. significant wear, corrosion or excessive discoloration from long term exposure to UV).

Advise the user to inform the practitioner/service provider of any changes in body weight and/or activity level.

If this device is used for extreme activity, the maintenance level and interval should be reviewed and if required advice and technical support sought to plan a new maintenance schedule dependent upon the frequency and nature of the activity. This should be determined by a local risk assessment carried out by a suitably qualified individual.

---

## 6 Limitations on Use

### Intended Life

A local risk assessment should be carried out based upon activity and usage.

### Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof to a maximum depth of 1 meter. Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts. Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water and debris ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).


Suitable for submersion

# 7 Bench Alignment

The instructions in this section are for practitioner use only.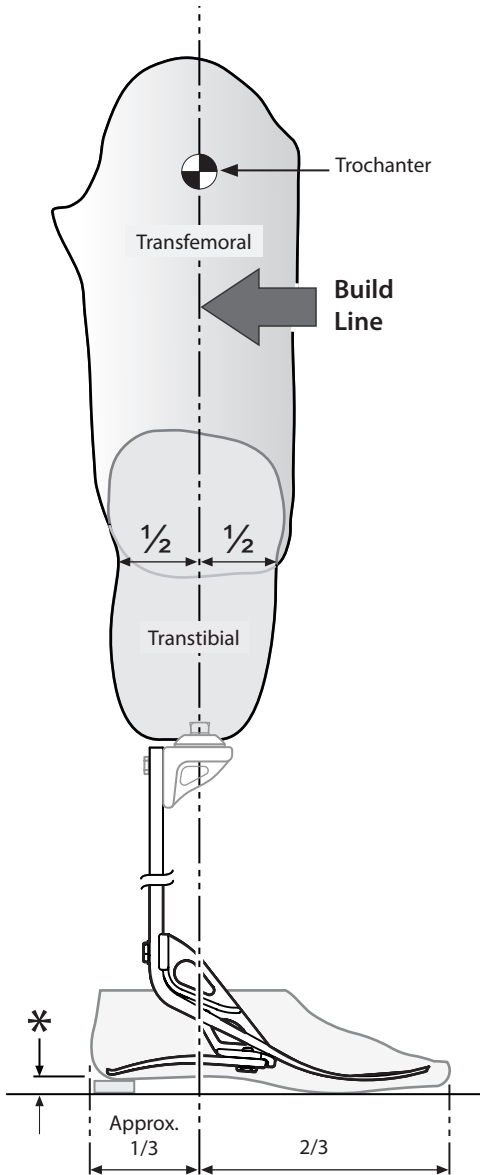

## 7.1 Static Alignment

### Set-up Length

Note: Shin blade should be vertical when used with a 10 mm heel height shoe. Lower heel height shoes may require the use of a packing piece below the heel.

Cut shin to approximate length, allowing 10–15 mm for final trimming. Note trial length will typically include 5 mm to allow for axial compression of heel and toe.

### Build Line

This should fall  $\frac{1}{3}$  of the foot length from the heel.

## 7.2 Dynamic Alignment

### Coronal Plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

### Sagittal Plane

Check for smooth transition from heel strike to toe-off. Ensure also that when standing the heel and toe are evenly loaded and that both are touching the floor.

## 7.3 Transfemoral Alignment

Align transfemoral components according to fitting instructions supplied with the knee, keeping the build line relative to this device, as shown.

\* Allow for user's own footwear

## 8 Cutting Pylon to Length



**The instructions in this section are for practitioner use only.**

Note: The following instructions assume the use of the male, sliding, rotating pyramid. If an alternative adapter is required, this should be taken into account throughout the build process.





-  **Use appropriate health and safety equipment at all times including extraction**
-  **Be aware of finger trap hazard at all times.**
-  **WARNING: Carbon fiber dust is non-toxic but is an irritant to the skin, eyes and lungs. Always read and comply with local and national Health & Safety regulations regarding the cutting of carbon fiber.**

**When cutting carbon fibre:**

1. Ensure working area is well ventilated and/or under extraction.
2. Wear eye protection/face shield, dust mask and protective gloves.

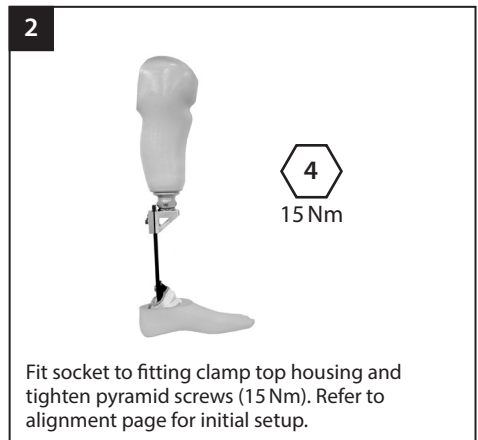
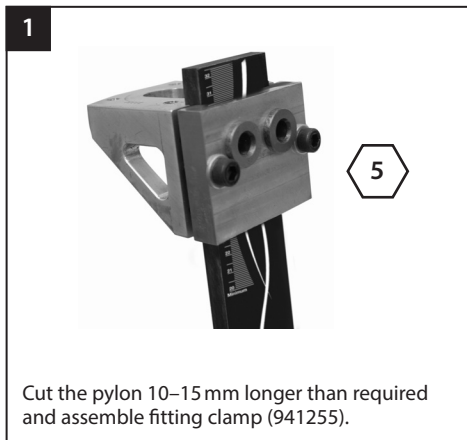


## 8 Cutting Pylon to Length *(continued)*

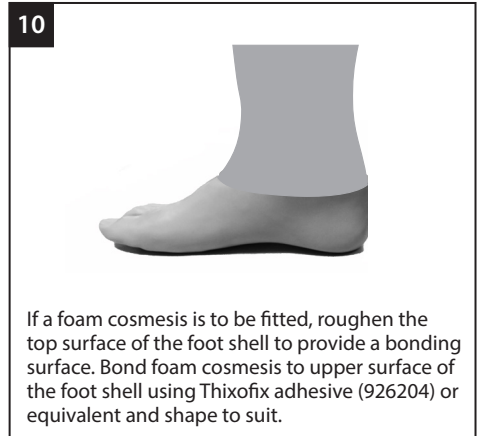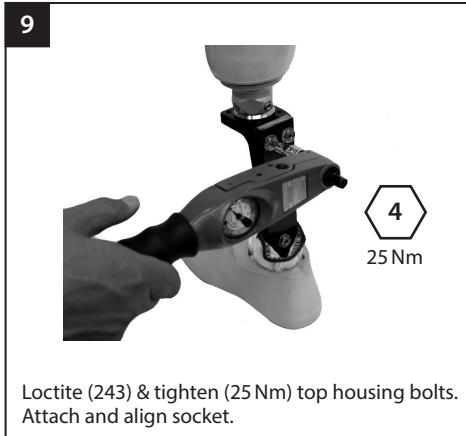
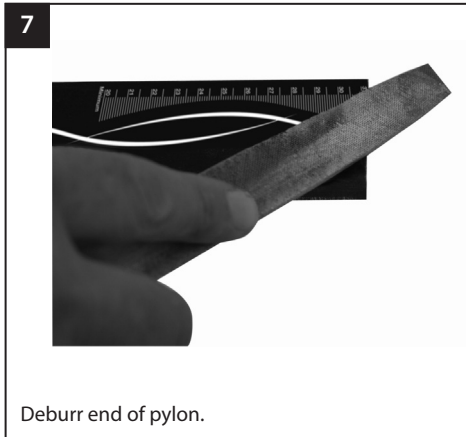
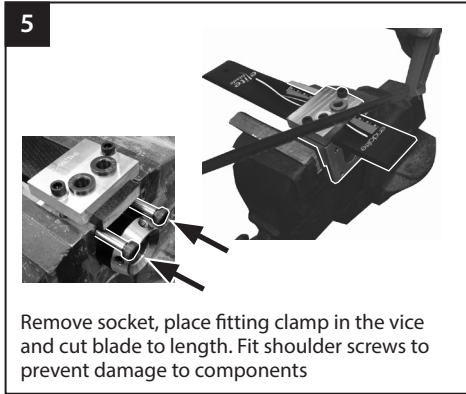
# 9 Foot Shell Removal & Pylon/Spring Replacement


*The instructions in this section are for practitioner use only.*

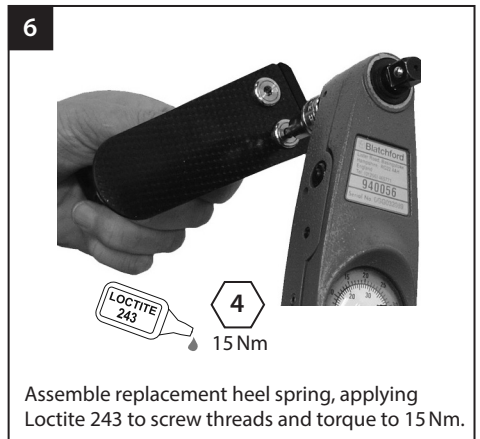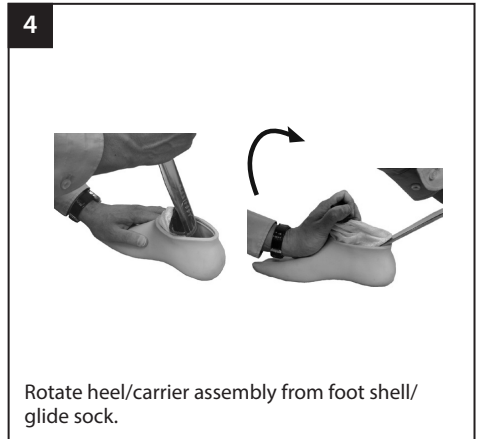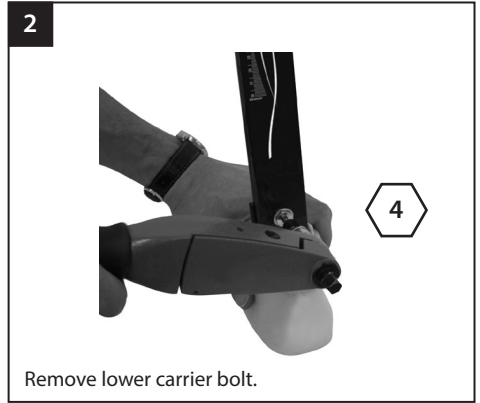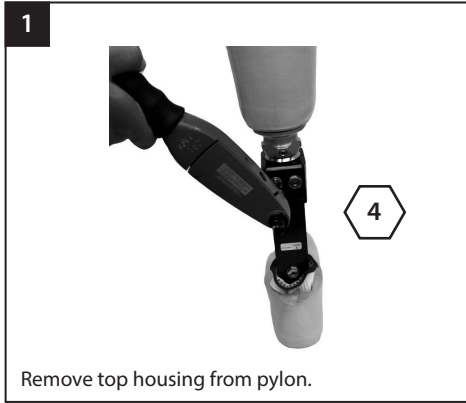## 9 Foot Shell Removal & Pylon/Spring Replacement (cont.)
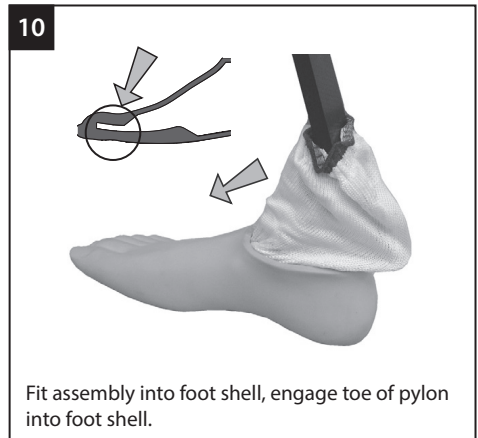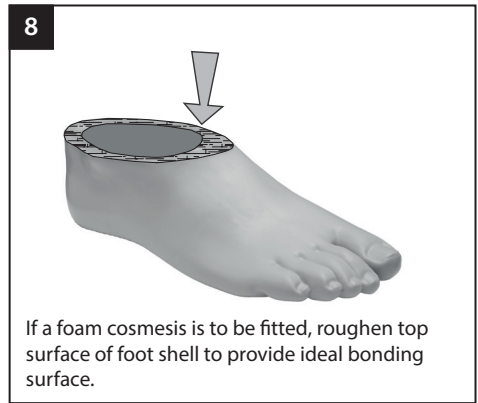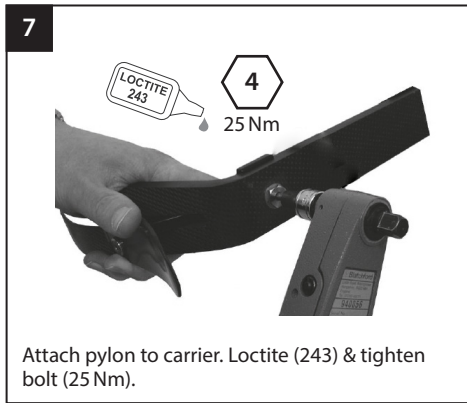
## 9 Foot Shell Removal & Pylon/Spring Replacement *(cont.)*
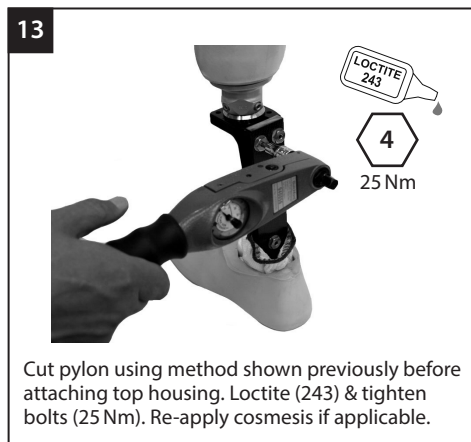
# 10 Fitting Advice

*The instructions in this section are for practitioner use only.*

Springs are supplied as matched sets i.e. the pylon/toe and heel springs are designed to work together to give smooth progression for most users.

## Static Alignment

Ensure flexion is fully accommodated. Adjust alignment to ensure the wearer can stand comfortably with the heel and forefoot evenly loaded.


|                           | Symptoms   | Remedy  |
|---------------------------|--|---|
| <b>Heel too soft</b>      | <ul style="list-style-type: none"><li>• Sinking at heel strike</li><li>• Difficulty climbing over the toe (toe feels too hard)</li></ul>   | Move socket anteriorly in relation to the foot (excess movement may result in drop off)   |
| <b>Heel too hard</b>      | <ul style="list-style-type: none"><li>• Rapid transition from heel strike through stance phase</li><li>• Difficulty in controlling heel action, foot jars into mid-stance</li><li>• Foot feels too rigid</li></ul> | Move socket posteriorly in relation to foot   |
| <b>Toe/Blade too soft</b> | <ul style="list-style-type: none"><li>• Rapid progression through mid-stance</li><li>• 'Drop off' at higher Activity Levels</li></ul>  | <ol style="list-style-type: none"><li>1. Move socket posteriorly in relation to foot</li><li>2. Plantarflex foot slightly - note: some re-alignment may be required</li></ol> |
| <b>Toe/Blade too hard</b> | <ul style="list-style-type: none"><li>• Difficulty in climbing over toe</li></ul>  | <ol style="list-style-type: none"><li>1. Move socket anteriorly in relation to foot</li><li>2. Dorsiflex foot slightly - note: some re-alignment may be required</li></ol>    |

Note... Please contact your sales representative or customer services if it is not possible to achieve a smooth gait after following the advice above.

# 11 Technical Data

|   |   |
|---|---|
| Operating and Storage Temperature Range:      | -15 °C to 50 °C<br>(5 °F to 122 °F)                     |
| Component Weight (size 26):                   | 670 g (1 lb 8 oz)                                       |
| Activity Level:                               | 3   |
| Maximum User Weight:                          | Levels 1–3: 166 kg (365 lb)<br>Level 4: 100 kg (220 lb) |
| Proximal Attachment Type:                     | Available with separately supplied alignment devices    |
| Build Height (excluding Top Housing Adapter): | 200–365 mm  |
| Heel Height:                                  | 10 mm   |

## Build Height

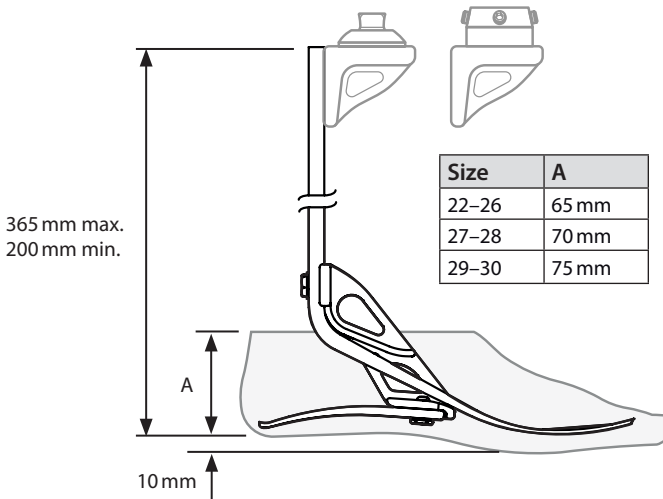## Storage and Handling

When storing for prolonged periods, ensure the product is free from moisture and stored at room temperature out of direct sunlight (protected from UV).

# 12 Ordering Information

## Order Example

|            |           |               |                 |                        |               |
|------------|-----------|---------------|-----------------|------------------------|---------------|
| <b>JAV</b> | <b>25</b> | <b>L</b>      | <b>N</b>        | <b>3</b>               | <b>S</b>      |
|            | Size      | Side<br>(L/R) | Width*<br>(N/W) | Spring Set<br>Category | Sandal<br>Toe |

Available from size 22 to size 30:

JAV22L1S to JAV30R9S

JAV22L1SD to JAV22L1SD

(add 'D' for a dark tone foot shell)

e.g. JAV25LN3S, JAV22R4S, JAV27RW4SD

\*Sizes 25-28 only. For all other sizes, omit the Width field.

## Spring Kits

| Rate  | Foot Sizes |            |           |                  |
|-------|------------|------------|-----------|------------------|
|       | Small (S)  | Medium (M) | Large (L) | Extra Large (XL) |
|       | 22-24      | 25-26      | 27-28     | 29-30            |
| Set 1 | 539301S    | 539510S    | 539519S   | Special Order    |
| Set 2 | 539302S    | 539511S    | 539520S   | Special Order    |
| Set 3 | 539303S    | 539512S    | 539521S   | 539530S          |
| Set 4 | 539304S    | 539513S    | 539522S   | 539531S          |
| Set 5 | 539305S    | 539514S    | 539523S   | 539532S          |
| Set 6 | 539306S    | 539515S    | 539524S   | 539533S          |
| Set 7 | 539307S    | 539516S    | 539525S   | 539534S          |
| Set 8 | 539308S    | 539517S    | 539526S   | 539535S          |
| Set 9 | -          | 539518S    | 539527S   | 539536S          |

## Foot Shell For dark add suffix D

| Size/Side | Narrow   | Wide     |
|-----------|----------|----------|
| 22L       | 539038S  | -        |
| 22R       | 539039S  | -        |
| 23L       | 539040S  | -        |
| 23R       | 539041S  | -        |
| 24L       | 539042S  | -        |
| 24R       | 539043S  | -        |
| 25L       | 539044SN | 539044SW |
| 25R       | 539045SN | 539045SW |
| 26L       | 539046SN | 539046SW |
| 26R       | 539047SN | 539047SW |
| 27L       | 539048SN | 539048SW |
| 27R       | 539049SN | 539049SW |
| 28L       | 539050SN | 539050SW |
| 28R       | 539051SN | 539051SW |
| 29L       | -        | 539052S  |
| 29R       | -        | 539053S  |
| 30L       | -        | 539054S  |
| 30R       | -        | 539055S  |

## Glide Sock

|             |        |
|-------------|--------|
| Sizes 22-26 | 531011 |
| Sizes 27-30 | 532811 |

## Fitting Clamp

|          |        |
|----------|--------|
| Part No. | 941255 |
|----------|--------|

## Accessories

|                    |        |
|--------------------|--------|
| Top Housing Male   | 539601 |
| Top Housing Female | 539602 |
| Foot Shell Cover   | 539087 |

## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

## CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: [www.blatchford.co.uk](http://www.blatchford.co.uk)

Medical Device

Single Patient – multiple use

## Compatibility

Combination with Blatchford branded products is approved based on testing in accordance with relevant standards and the MDR including structural test, dimensional compatibility and monitored field performance.

Combination with alternative CE marked products must be carried out in view of a documented local risk assessment carried out by a Practitioner.

## Warranty

This device is warranted for for 36 months - foot shell 12 months - glide sock 3 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See the Blatchford website for the current full warranty statement.

## Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

## Environmental Aspects

This product is made from a recyclable material. Where possible, the components should be recycled in accordance with local waste handling regulations.

## Retaining the Packaging Label

The practitioner is advised to keep the packaging label as a record of the device supplied.

## Trademark Acknowledgements

Blatchford is a registered trademark of Blatchford Products Limited.

## Manufacturer's Registered Address

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**Blatchford Europe GmbH**
**Endolite India Ltd.**