Elite2

User Guide

EL22L1S-EL30R9S
EL22L1SD-EL30R9SD

EN	User Guide	2
SL	Uporabniški priročnik	8
BG	Наръчник на потребителя	14
HR	Korisnički priručnik	20
SK	Používateľská príručka	26
HU	Felhasználói útmutató	32
EL	Οδηγός χρήσης	38
LV	Lietotāja rokasgrāmata	44
LT	Naudotojo vadovas	50
ET	Kasutaja käsiraamat	56

Contents

Contents	2
1 Description and Intended Purpose	3
2 Safety Information	3
3 Construction	4
4 Maintenance	5
5 Limitations on Use	5
6 Technical Data	6

1 Description and Intended Purpose

Please keep these instructions.

The term device is used throughout this document to refer to Elite2.

Application

This device is to be used exclusively as part of a lower limb prosthesis.

Intended for a single user.

A high-energy-return foot. The independent heel and toe springs provide axial deflection. The split toe provides good ground compliance.

2 Safety Information

This warning symbol highlights important safety information which must be followed carefully.

Any changes in the performance or function of the limb e.g. restricted or excessive movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

The device is not suitable for extreme sports, competitive running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling or running is acceptable.

Assembly, maintenance and repair of the device must only be carried out by a suitably qualified clinician.

The user should contact their clinician if their condition changes.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.




To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

3 Construction

Principal Parts:

- Carrier Assembly with Pyramid (aluminum/ stainless steel/titanium)
- Heel & Toe Springs (e-Carbon)
- Spring Attachment Screws (titanium)
- Glide Sock (UHM PE)
- Foot Shell (PU)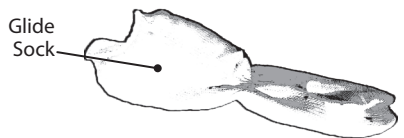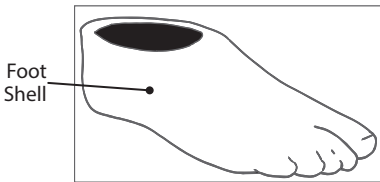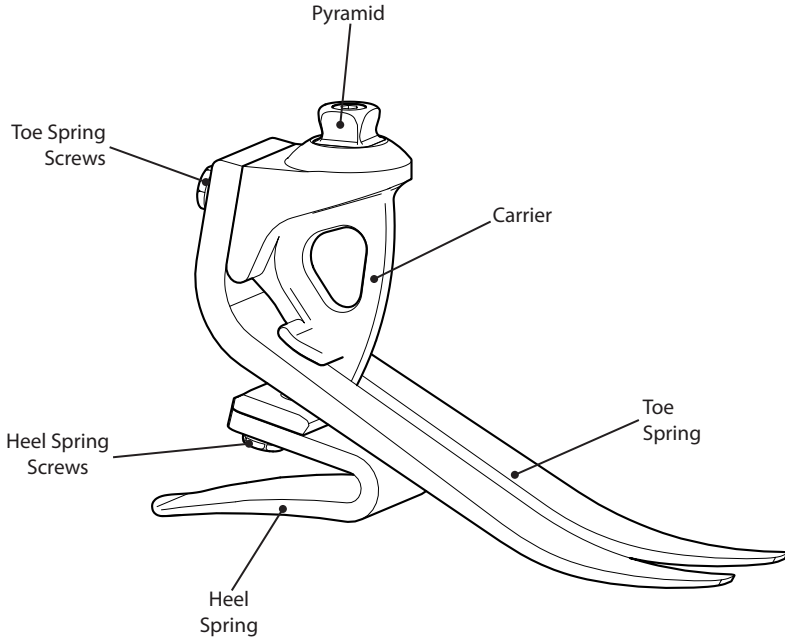

4 Maintenance

Maintenance must be carried out by competent personnel.

We recommend a regular visual check at least annually. Signs of wear that may affect function should be reported to their service provider (e.g. significant wear or excessive discoloration from long term exposure to UV).

If this device is used for extreme activity, the maintenance level and interval should be reviewed and if required advice and technical support sought to plan a new maintenance schedule dependent upon the frequency and nature of the activity. This should be determined by a local risk assessment carried out by a suitably qualified individual.

Any changes in performance of this device must be reported to the practitioner

Changes in performance may include:

- reduced energy return
- unusual noises
- excessive movement

The practitioner must also be informed of any changes in body weight and/or activity level.

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

5 Limitations on Use

Lifting Loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Suitable for submersion


Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, it should be inverted and dried before further use.

Exclusively for use between -15 °C and 50 °C.

6 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight (size 26):	680 g (1 lb 7 oz)
Activity Level:	3-4
Maximum User Weight:	166 kg (365 lb)
Proximal Alignment Attachment:	Male Pyramid (Blatchford)
Range of Adjustment:	±7° Angular
Build Height: (See diagram below)	130 mm
Heel Height:	10 mm

Fitting Length

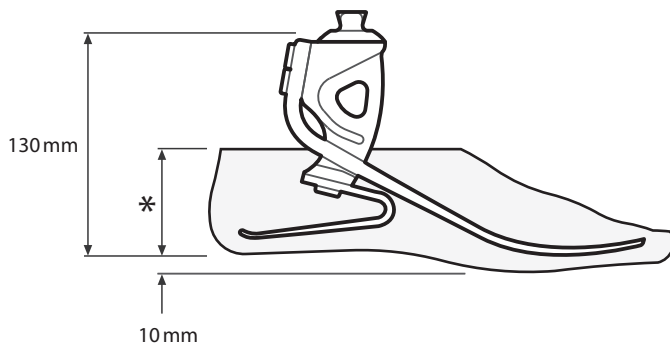* Sizes
22-26 = 65mm
27-28 = 70mm
29-30 = 75mm

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Medical Device

Single Patient – multiple use

Warranty

This device is warranted for 36 months - foot shell 12 months - glide sock 3 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

See Blatchford website for the current full warranty statement.

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH.

Trademark Acknowledgements

Elite Foot and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford Europe GmbH
Endolite India Ltd.